


THUNDERBEAT & SONICBEAT Assembly Guide

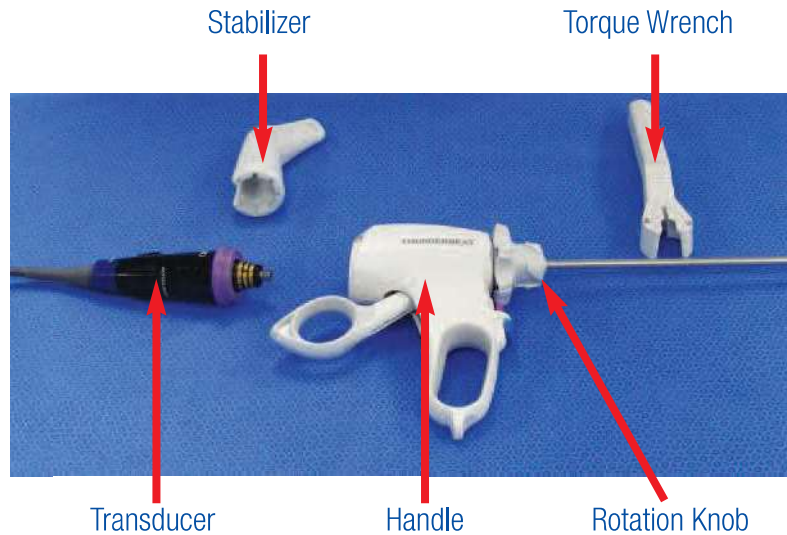

1 Insert the Thunderbeat or Sonicbeat transducer into the handle.



2 While holding the transducer, turn the rotation knob clockwise until it stops. **Do not rotate the transducer or the handle.**



3 Mount the stabilizer on the transducer.



4 Attach the head of the torque wrench to the rotation knob.



5 While holding only the stabilizer (**not the handle**) slowly turn the torque wrench clockwise until you hear a click. The transducer is now firmly in place. Remove the torque wrench and stabilizer.



6 For disassembling the instrument and transducer, use the torque wrench to turn the rotation knob counterclockwise while holding the stabilizer.



* Some devices are not available in some areas.

** This sheet is for your reference only. Please refer to the instructions provided with the device for more precise instructions.