

EZDilate™ Endoscopic Balloon Dilator Quick Reference Guide

The EZDilate is designed to dilate strictures in the esophagus, pylorus, papilla and colon.

CAUTION: This guide is only a summary of the instructions for use for EZDilate Endoscopic Balloon Dilator. Be sure to follow the Olympus EZDilate Endoscopic Balloon Dilator instruction manual that was included in the device package.

EZDilate Endoscopic Balloon Dilator Preparation



- 1 Ensure inflator plunger is in the unlocked position by pushing the locking mechanism to the left.



- 2 Fill the inflator with 40 mL of water or contrast. Make sure the OFF knob of the stopcock is in the 3 o'clock position.

Note: It is important that you do not overfill the inflator to ensure an adequate vacuum is created to deflate the balloon after dilation.



- 3 Remove air from the inflator.
 - Turn the inflator so that the pressure gauge is facing the ceiling and any air rises to the top of the inflator. (See image.)
 - Push the plunger until a steady stream of contrast comes out of the stop cock. Be sure to remove all air bubbles from the syringe.

EZDilate Endoscopic Balloon Dilator Quick Reference Guide


The EZDilate is designed to dilate strictures in the esophagus, pylorus, papilla and colon.

CAUTION: This guide is only a summary of the instructions for use for EZDilate Endoscopic Balloon Dilator. Be sure to follow the Olympus EZDilate Endoscopic Balloon Dilator instruction manual that was included in the device package.

EZDilate Endoscopic Balloon Dilator Preparation (continued)



- 4** Remove air from the balloon.
- Connect the balloon catheter to the stopcock by twisting the two together. (See image A.)
 - Turn the inflator upside down so that the pressure gauge is pointed towards the ground. Pull the plunger back as far as possible and lock it in place by pushing the locking mechanism to the center position. (See image B.)
 - Turn the inflator so that the pressure gauge is facing the ceiling, allowing any air to rise to the top of the inflator. (See image C.)
 - Turn the OFF knob of the stopcock to 12 o'clock position. You will hear the air being released. (See image D.)

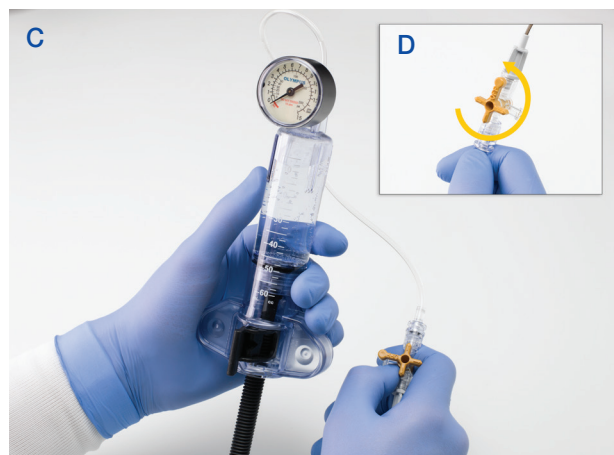

- 5**
- With the air at the top of the inflator, unlock the plunger by pushing the locking mechanism to the left (See image A.) and push the plunger up to remove any additional air. Continue to push until a steady stream of water or contrast comes out of the stopcock. (See image B.)

EZDilate™ Endoscopic Balloon Dilator Quick Reference Guide

The EZDilate is designed to dilate strictures in the esophagus, pylorus, papilla and colon.

CAUTION: This guide is only a summary of the instructions for use for EZDilate Endoscopic Balloon Dilator. Be sure to follow the Olympus EZDilate Endoscopic Balloon Dilator instruction manual that was included in the device package.

EZDilate Endoscopic Balloon Dilator Preparation (continued)



- 6** Return the OFF knob of the stopcock to the 3 o'clock position. The EZDilate balloon is now primed for use.

Inflate the EZDilate Endoscopic Dilation Balloon



- 1** Unlock the inflator by pushing the locking mechanism to the left.



- 2** Push the plunger in until you meet resistance. Lock the plunger in place by moving locking mechanism to center position.

EZDilate Endoscopic Balloon Dilator Quick Reference Guide

The EZDilate is designed to dilate strictures in the esophagus, pylorus, papilla and colon.

CAUTION: This guide is only a summary of the instructions for use for EZDilate Endoscopic Balloon Dilator. Be sure to follow the Olympus EZDilate Endoscopic Balloon Dilator instruction manual that was included in the device package.

Inflate the EZDilate Endoscopic Dilation Balloon (continued)



3 Turn the plunger handle clockwise until you have reached the atm pressure required to reach the desired balloon diameter. Use the pressure/balloon diameter guide attached to the catheter of the balloon.

Deflate the EZDilate Endoscopic Dilation Balloon



1 Unlock the inflator by pushing the locking mechanism to the left.



2 Grab the inflator with your left hand, palm facing down and right hand on the plunger, thumb on the locking mechanism.

EZDilate™ Endoscopic Balloon Dilator Quick Reference Guide

The EZDilate is designed to dilate strictures in the esophagus, pylorus, papilla and colon.

CAUTION: This guide is only a summary of the instructions for use for EZDilate Endoscopic Balloon Dilator. Be sure to follow the Olympus EZDilate Endoscopic Balloon Dilator instruction manual that was included in the device package.

Deflate the EZDilate Endoscopic Dilation Balloon (continued)



3 Pull the plunger all the way back and relock the inflator with your left thumb.



4 You may now let go of the inflator and let the balloon deflate.

Olympus and EZDilate are trademarks or registered trademarks of Olympus Corporation, Olympus America Inc., and/or their affiliates.