



## HiQ+ Laparoscopic Instruments Assembly Guide

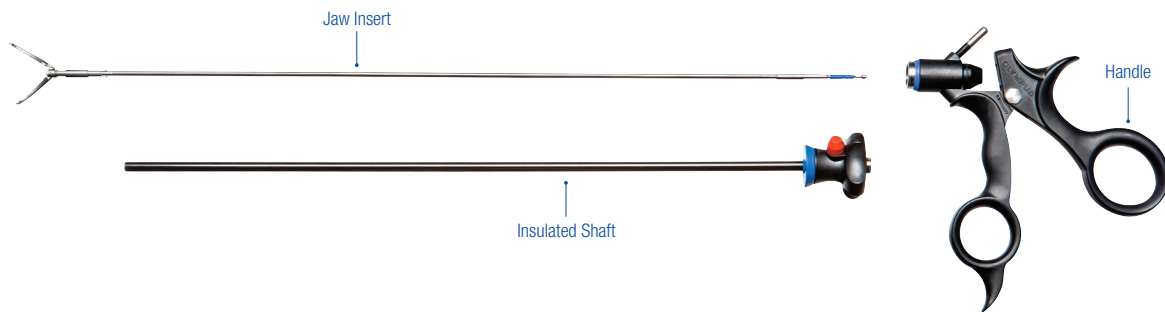

**HiQ+ Components:** The HiQ+ Monopolar Laparoscopic Hand Instruments consists of three modular components and 2 sealing caps.


### STEP 1

Insert the jaw insert into the insulated shaft

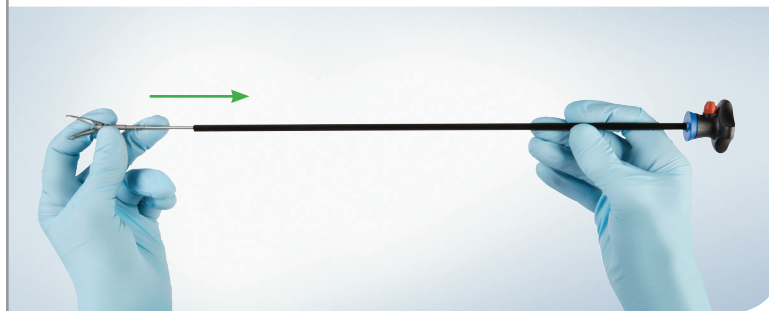


### STEP 2

Anchor the jaw insert with your left hand then rotate the shaft clockwise a half-turn to lock the jaw insert in place.

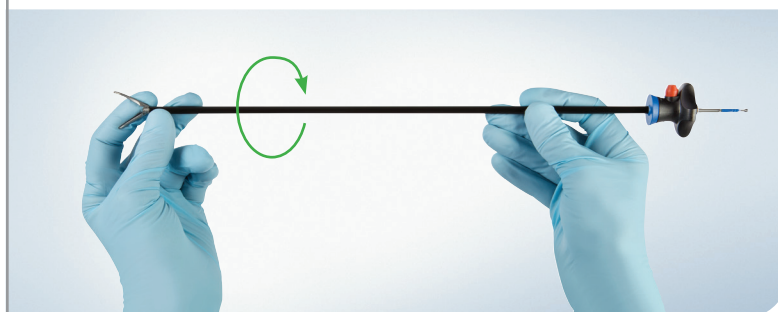

### STEP 3

Completely open the handle.

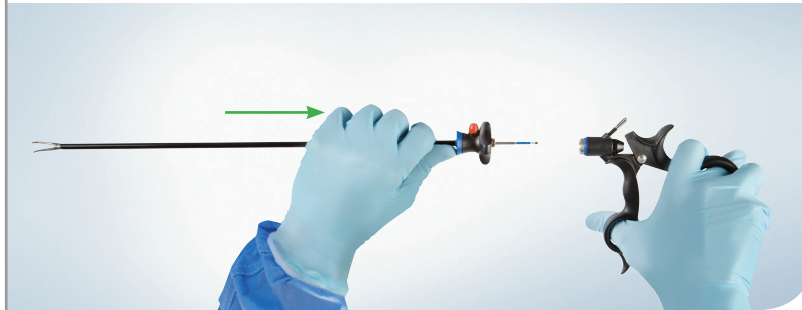

### STEP 4

Fully insert the proximal end of the jaw insert into the handle. If the jaw insert becomes dislodged from the shaft, disassemble all components and restart the assembly process.



### STEP 5

When properly inserted, close the handle to lock the system in place.



Olympus is a registered trademark of Olympus Corporation, Olympus America Inc., and/or their affiliates.