

## HiQ+ Laparoscopic Instruments Disassembly Guide


### STEP 1

Completely open the handle.



### STEP 2


With the handle fully opened, press down on the red release button.

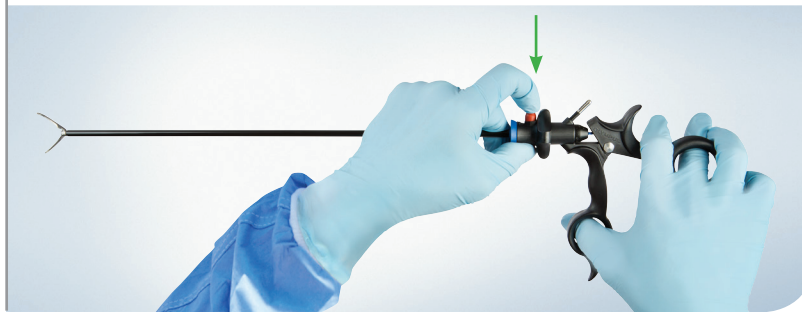

### STEP 3


Pull apart the jaw insert and shaft component from the handle.

*NOTE: A twisting motion may help facilitate the separation of the two components.*

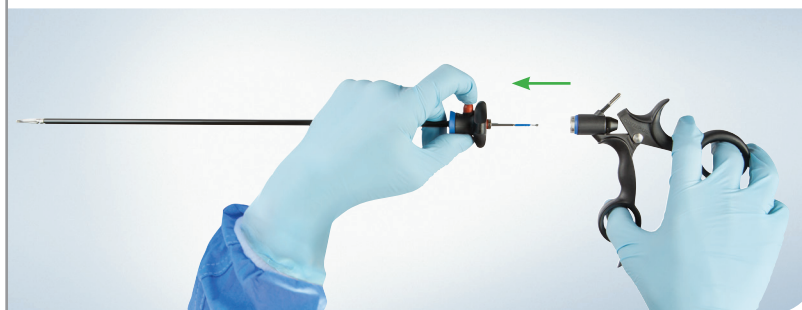

#### STEP 4

Anchor the jaw insert with your left hand then rotate the shaft counter-clockwise a half-turn to dislodge the jaw insert from the insulated shaft.

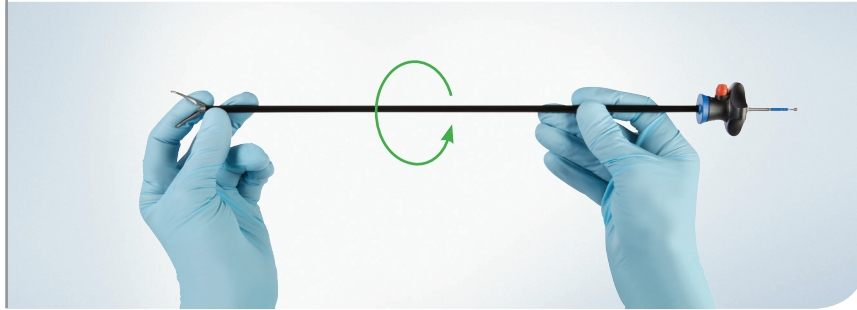

#### STEP 5


Remove the jaw insert from the insulated shaft.

*NOTE: Push down on the red release button if disassembling bipolar instruments.*



#### STEP 6

Remove the red sealing cap on the insulated shaft and the black sealing cap from the handle. Prepare for cleaning and sterilization.

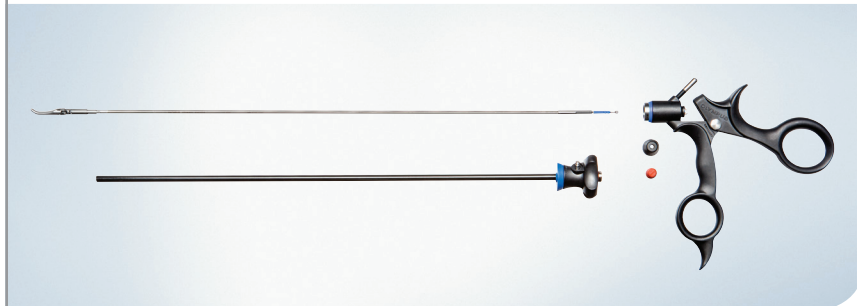

Olympus is a registered trademark of Olympus Corporation, Olympus America Inc., and/or their affiliates.