


ENDOSCOPE CLEANING GUIDE

for EVIS X1 Gastro and Colonovideoscope

Step 1 to 10 of cleaning for endoscope are done in detergent solution or water completely, except for flushing with air and drying the external surfaces at the end of cleaning.

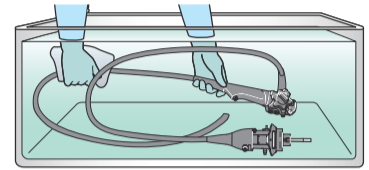

WARNING



This workflow is **a summary** of the cleaning steps necessary to properly reprocess your EVIS X1 Gastro and Colonovideoscope and does not include disinfecting/sterilizing steps. Follow the detailed steps instructed in the latest **ENDOSCOPE REPROCESSING MANUAL** for EVIS X1 Gastro and Colonovideoscope.

* The channel-opening cleaning brush part of the single use combination (BW-412T)

** The channel cleaning brush part of the single use combination cleaning brush (BW-412T)

Clean the external surfaces	Brush the instrument channel and suction channel	Brush the suction cylinder & the instrument channel port	Aspirate detergent solution
1 Detergent solution Before "Cleaning the external surfaces", "Precognizing" and "Leakage testing" should be done as described in the reprocessing manual. 	2 Detergent solution 	3 Detergent solution 	4 Detergent solution
5 Detergent solution 		6 Detergent solution Flush the air/water channel with detergent solution 	
7 Detergent solution Flush the auxiliary water channel with detergent solution (only applicable to endoscope with auxiliary water feeding) 		8 Detergent solution 	
9 Water Remove detergent solution from all channels 			
10 Water 			
11 Air 			12 Dry the external surfaces

Manufactured by
OLYMPUS MEDICAL SYSTEMS CORP.
 2951 Ishikawa-cho, Hachioji-shi, Tokyo 192-8507, Japan
 Fax: (042)646-2429 Telephone: (042)642-2111

Distributed by
OLYMPUS EUROPA SE & CO. KG
 (Premises/Goods delivery) Wendenstrasse 14-18, 20097 Hamburg, Germany
 (Letters) Postfach 10 49 08, 20034 Hamburg, Germany
 Fax: (040)23773-4656 Telephone: (040)23773-0

KEYMED (MEDICAL & INDUSTRIAL EQUIPMENT) LTD.
 KeyMed House, Stock Road, Southend-on-Sea, Essex SS2 5QH, United Kingdom
 Fax: (01702)465677 Telephone: (01702)616333

OLYMPUS MOSCOW LIMITED LIABILITY COMPANY
 Elektrozavodskaya str. 27 bld.8, 107023 Moscow, Russia
 Fax: (7495)926-7072 Telephone: (7495)926-7077

OLYMPUS AMERICA INC.
 3500 Corporate Parkway, P.O. Box 610, Center Valley, PA 18034-0610, U.S.A.
 Fax: (484)398-7129 Telephone: (484)398-5000

OLYMPUS LATIN AMERICA INC.
 5301 Blue Lagoon Drive, Suite 290 Miami, FL 33126-2097, U.S.A.
 Fax: (305)261-4421 Telephone: (305)266-2332

OLYMPUS (BEIJING) SALES & SERVICE CO., LTD.
 A6F, Ping An International Financial Center, No. 1-3, Xinyuan South Road,
 Chaoyang District, Beijing, 100027 P.R.C.
 Fax: (86)10-5976-1299 Telephone: (86)10-5819-9000

OLYMPUS AUSTRALIA PTY LTD
 3 Acacia Place, Notting Hill, VIC 3168, Australia
 Fax: (03)9543-1350 Telephone: (03)9265-5400

OLYMPUS SINGAPORE PTE LTD
 436B Alexandra Road #03-07/12, Alexandra Technopark Bk B, Singapore 119968
 Fax: 6834-2438 Telephone: 6834-0010

OLYMPUS KOREA CO., LTD.
 2F-3F Majestor City Tower One, 12, Seocho-daero 38-gil, Seocho-gu, Seoul,
 06655, Korea
 Fax: (02)6255-3210 Telephone: 1544-3202