

## ORBEYE System Set-up

### Step 1



Press Tower Power Button On

### Step 2



System Boots Up

### Step 3



Main Menu Screen Comes On

### Step 4



Place White Paper/Gauze Under Head

### Step 5



Press and Hold White Balance for 2 Seconds

### Step 6



Beeps and Says Completed

### Step 7



Press Load User Preset

### Step 8



Select Case Type


### Step 9



Home Screen Appears

# Monitor Set-Up

## Step 1

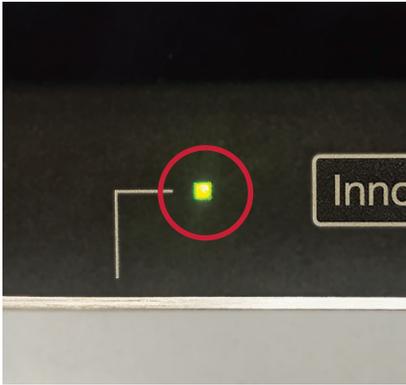


Turn On Monitor

## Step 2



Press Control to Access Menu

## Step 3

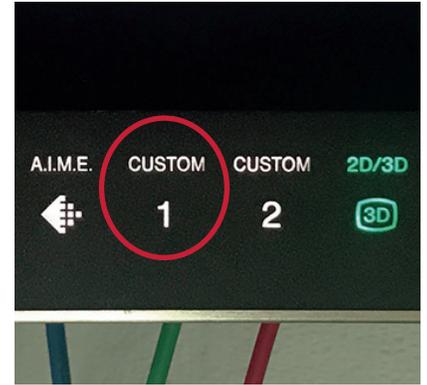


Press Custom 1

## Step 4



Use 2 Arrows to Select Flip

## Step 5

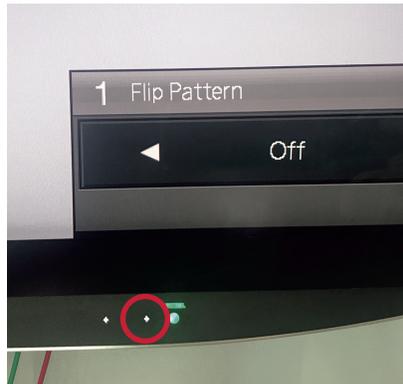


Use Flip (OFF) for Main ORBEYE Monitor

## Step 6



Use Flip (Rotation) for Assistant Monitor

## Step 7

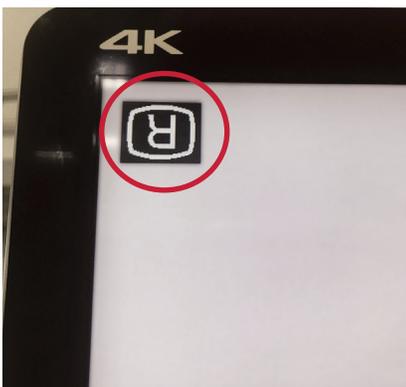


Rotation is On When (R) is Upside Down

## Step 8



Connect Each Monitor Cable  
White Dot Up

## Step 9

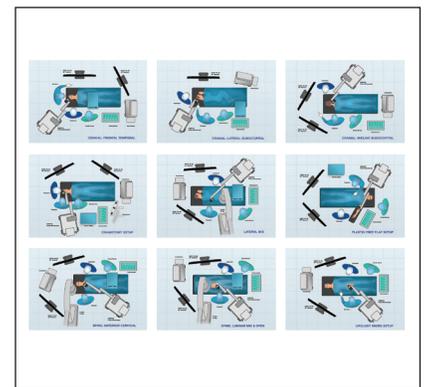

Set Up Room by Case Needs  
[Refer to Room Setup Guide](#)

# Head Controls

## Step 1



Trigger Brake Release Pull To Move Release to Lock

## Step 2



A – Up and Down Movement Focuses ORBEYE

## Step 3



B – Up and Down Movement Zooms ORBEYE

## Step 4



Button (1)  
Brightness Up (+)

## Step 5



Button (2)  
Brightness Down (-)

## Step 6



Depth of Field  
(1 -5) Cycles Through

**Buttons 1, 2 and 3 can be programmed with different functions and saved by user profile.**

