

Reprocessing Flexible Endoscope Accessories Manual Cleaning Prior to Sterilization for the MAJ-891



Assembled MAJ-891 (Forceps/Irrigation Plug)

DISCLAIMER



This sheet is for quick reference only. Please refer to the relevant User's Manual(s) for instructions, warnings and cautions. Nothing in this presentation is meant to supersede or replace the instructions for use applicable to each specific device, or the processes and procedures in place at your facility.

WARNING



Be sure to disassemble the forceps/irrigation plug before it is cleaned, disinfected or sterilized. Otherwise, the forceps/irrigation plug may not be properly cleaned/disinfected/sterilized.



Disassembled MAJ-891 (Forceps/Irrigation Plug)

Precleaning

1 Disconnect from the endoscope




2 Completely immerse in detergent solution



Manual Cleaning

3 Completely Disassemble

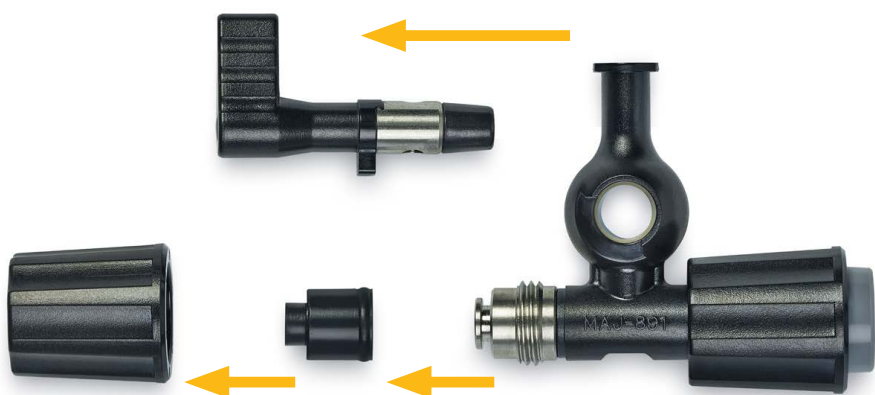


4 Completely immerse in detergent solution



Manual Cleaning


5 Thoroughly wipe all external surfaces and brush all internal surfaces of each part.

Note: to pass the brush through the biopsy valve, press it between your fingers.
Thoroughly rinse after manual cleaning.

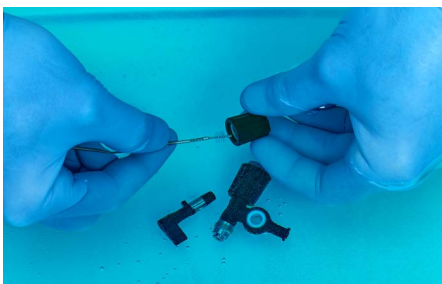

Drying

6 Use a lint-free cloth to thoroughly wipe and dry the external surfaces of each part.

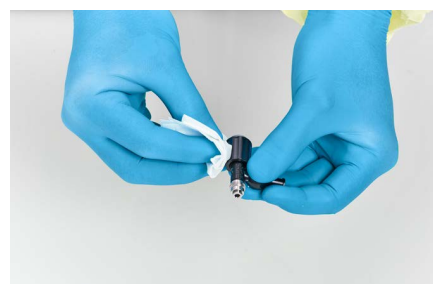

7 Use sterile cotton swabs to dry the internal surfaces of each part.



Note: Prior to sterilization, the instrument must be thoroughly cleaned and dried. All parts must be sealed in packages separately in packaging appropriate for the sterilization method chosen.

