

Reprocessing Flexible Endoscope Accessories

Manual Cleaning and High Level Disinfection for the MAJ-891



Assembled MAJ-891 (Forceps/Irrigation Plug)

WARNING This workflow is a **SUMMARY** of the reprocessing steps necessary to properly reprocess your MAJ-891. **Following disconnection and disassembly of the MAJ-891 utilize the IFU for how to reprocess the MAJ-891.**



WARNING Be sure to disassemble the forceps/irrigation plug before it is cleaned, disinfected or sterilized. Otherwise, the forceps/irrigation plug may not be properly cleaned/disinfected/sterilized.



Disassembled MAJ-891 (Forceps/Irrigation Plug)

Precleaning

1 Disconnect from the endoscope




2 Completely immerse in detergent solution



Manual Cleaning

3 Completely Disassemble

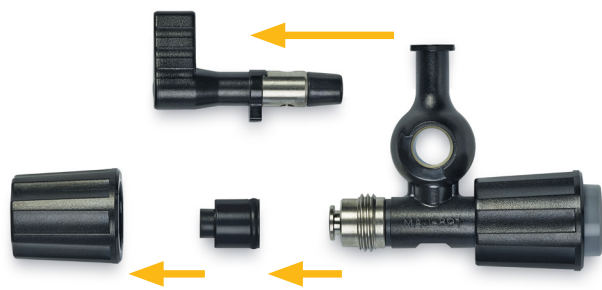

4 Completely immerse in detergent solution



5 Thoroughly wipe all external surfaces and brush all internal surfaces of each part



Note: to pass the brush through the biopsy valve, press it between your fingers.

Thoroughly rinse after manual cleaning.



Manual Disinfection


6 Completely immerse all parts in the disinfectant solution



7 Fill a 30 ml syringe with disinfectant solution and flush the forceps/irrigation plug by covering the stopcock's holes with your fingers and fill the channel with disinfectant solution



8 Open the slit of the biopsy valve in disinfectant solution

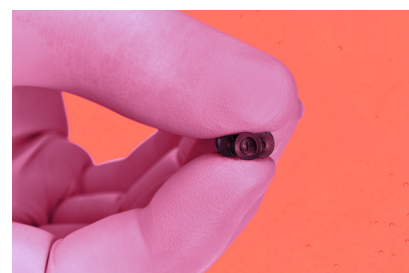

9 Cover the basin and soak



Rinsing

10 Immerse all parts in sterile water. Flush the forceps/irrigation plug utilizing a 30ml syringe.




Note: when non-sterile water is used after disinfection, immerse all parts in 70% ethyl or isopropyl alcohol and gently agitate them.

Drying

11 Wipe and dry the external surfaces with sterile, lint-free cloth



12 Use sterile cotton swabs to dry the internal surfaces of each part

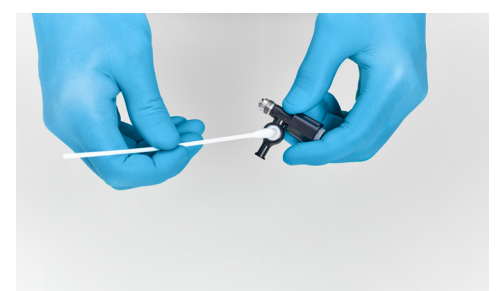

Reprocessing Flexible Endoscope Accessories Manual Cleaning Prior to Sterilization for the MAJ-891



Assembled MAJ-891 (Forceps/Irrigation Plug)

WARNING This workflow is a **SUMMARY** of the reprocessing steps necessary to properly reprocess your MAJ-891. **Following disconnection and disassembly of the MAJ-891 utilize the IFU for how to reprocess the MAJ-891.**



WARNING Be sure to disassemble the forceps/irrigation plug before it is cleaned, disinfected or sterilized. Otherwise, the forceps/irrigation plug may not be properly cleaned/disinfected/sterilized.



Disassembled MAJ-891 (Forceps/Irrigation Plug)

Precleaning

1 Disconnect from the endoscope




2 Completely immerse in detergent solution



Manual Cleaning

3 Completely Disassemble

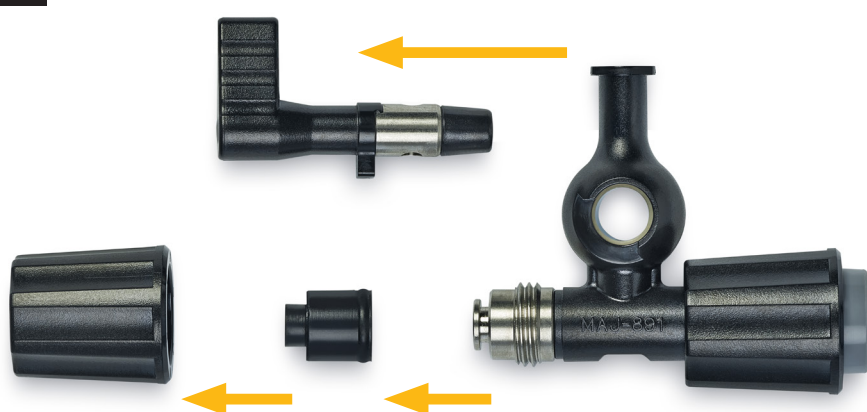



4 Completely immerse in detergent solution



Manual Cleaning

5 Thoroughly wipe all external surfaces and brush all internal surfaces of each part.

Note: to pass the brush through the biopsy valve, press it between your fingers.
Thoroughly rinse after manual cleaning.

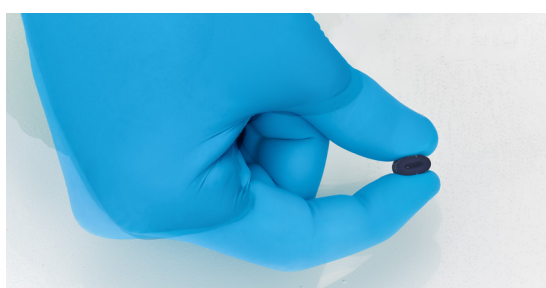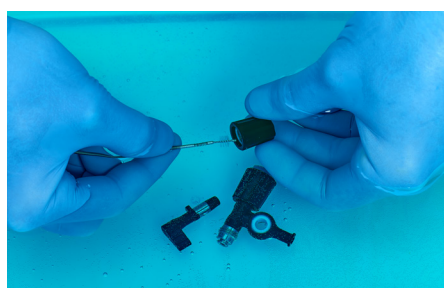

Drying

6 Use a lint-free cloth to thoroughly wipe and dry the external surfaces of each part.



7 Use sterile cotton swabs to dry the internal surfaces of each part.



Note: Prior to sterilization, the instrument must be thoroughly cleaned and dried. All parts must be sealed in packages separately in packaging appropriate for the sterilization method chosen.