

ENDOSCOPE PRECLEANING GUIDE

for EVIS X1, LUCERA, EXERA Gastro & Colonovideoscope

WARNING



This workflow is a **summary** of the precleaning steps necessary in reprocessing EVIS X1 Gastro and Colonovideoscopes. This should be conducted immediately after a patient procedure. It does not include **leakage testing/manual cleaning/disinfecting/sterilizing** steps. Follow the detailed steps as instructed in the latest **ENDOSCOPE REPROCESSING MANUAL** for EVIS X1, LUCERA, EXERA GASTRO & COLONOVIDEOSCOPE.



1 Wipe down the insertion section with water* soaked clean lint-free cloths (single-use)



2 Depress the suction valve to aspirate water for 10 seconds or more, through the instrument /suction channel



3 Depress the suction valve to aspirate air for 10 seconds or more, through the instrument /suction channel



4 Attach AW channel cleaning adapter. Depress the button of AW channel cleaning adapter to flush the air channel with water for 10 seconds or more.



5 Then release the button to flush the air/water channel with air for 10 seconds or more.



4 Option 1 Flush 30ml of water* into the auxiliary water channel



4 Option 2 Alternative method of Step 4 : Use OFP-2 to flush the auxiliary water channel with water for a minimum of 10 seconds, with maximum flow rate.



5 Detach the suction tube and the water container's metal tip from the endoscope, then detach the endoscope connector from the video system centre.

KEY	
	Water
	Air

* If necessary, a detergent solution can be used instead of water.

Manufactured by
OLYMPUS MEDICAL SYSTEMS CORP.
 2951 Ishikawa-cho, Hachioji-shi, Tokyo 192-8507, Japan
 Fax: (042)646-2429 Telephone: (042)642-2111

Distributed by
OLYMPUS EUROPA SE & CO. KG
 (Premises/Goods delivery) Wendenstrasse 14-18, 20097 Hamburg, Germany
 (Letters) Postfach 10-40 08, 20034 Hamburg, Germany
 Fax: (040)23773-4656 Telephone: (040)23773-0

KEYMED (MEDICAL & INDUSTRIAL EQUIPMENT) LTD.
 KeyMed House, Stock Road, Southend-on-Sea, Essex SS2 5QH, United Kingdom
 Fax: (01702)465877 Telephone: (01702)619333

OLYMPUS MOSCOW LIMITED LIABILITY COMPANY
 Elektrovozdskaya str. 27 bld 8, 101023 Moscow, Russia
 Fax: (7495-926-7072 Telephone: (7495-926-7077

OLYMPUS AMERICA INC.
 3520 Corporate Parkway, P.O. Box 610, Center Valley, PA 18034-0610, U.S.A.
 Fax: (484)896-7126 Telephone: (484)996-5000

OLYMPUS LATIN AMERICA, INC.
 5301 Blue Lagoon Drive, Suite 250 Miami, FL 33126-2097, U.S.A.
 Fax: (305)261-4421 Telephone: (305)266-2332

OLYMPUS (BEIJING) SALES & SERVICE CO., LTD.
 ADF Ping An International Financial Center, No. 1-3, Xinyuan South Road,
 Chaoyang District, Beijing, 100027 P.R.C.
 Fax: (86)10-5976-1299 Telephone: (86)10-5819-9000

OLYMPUS AUSTRALIA PTY LTD
 3 Acacia Place, Notting Hill, VIC 3168, Australia
 Fax: (03)9543-1350 Telephone: (03)9545-5400

OLYMPUS SINGAPORE PTE LTD
 438B Alexandra Road #03-07/1, Alexandra Technopark Bldg B, Singapore 119968
 Fax: 6834-2438 Telephone: 6834-0010

OLYMPUS KOREA CO., LTD.
 2F-3F Majeon City Tower One, 12, Seocho-daero 38-gil, Seocho-gu, Seoul,
 06655, Korea
 Fax: (02)6255-3210 Telephone: 1544-3202