

EndoClot® Submucosal Injection Solution (SIS)

Quick Reference Guide

CAUTION: This guide is only a summary of the instructions for use for the EndoClot® Submucosal Injection Solution. Be sure to follow the instruction manual included in the device packaging.

Product inspection and components

- Inspect package for damage
- Peel off cover



- Spiral dispensing syringe
- Vial containing 0.5g (SIS-15)/1g (SIS-30) AMP® powder
- Plastic tray with holder for vial



Prepare the agent



1. Place vial in holder.



2. Inject 15ml (SIS-15) /30ml (SIS-30) of sterile 0.9% saline into vial.



3. Mix by shaking vial.

EndoClot® Submucosal Injection Solution (SIS)

Quick Reference Guide

CAUTION: This guide is only a summary of the instructions for use for the EndoClot® Submucosal Injection Solution. Be sure to follow the instruction manual included in the device packaging.

Prepare the agent (continued)



4. Attach needle to SIS syringe.

Draw the agent into syringe



5. Unscrew locking cap to free syringe plunger.

Draw the agent into syringe (continued)



6. Pull the syringe plunger to draw agent.



7. Remove needle from syringe, then purge any air in the syringe.



8. Re-screw the locking cap.

EndoClot® Submucosal Injection Solution (SIS)


Quick Reference Guide

CAUTION: This guide is only a summary of the instructions for use for the EndoClot® Submucosal Injection Solution. Be sure to follow the instruction manual included in the device packaging.

Connect injection needle



9. Connect to needle and tighten the Luer-lock.

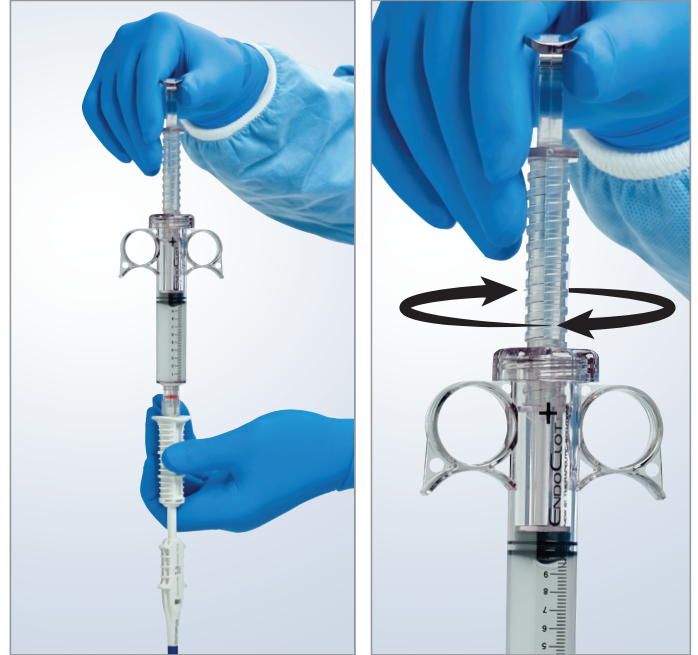


10. Purge air from needle by winding plunger.

Deploy



11. Pass the injection needle down the scope channel.

Injection

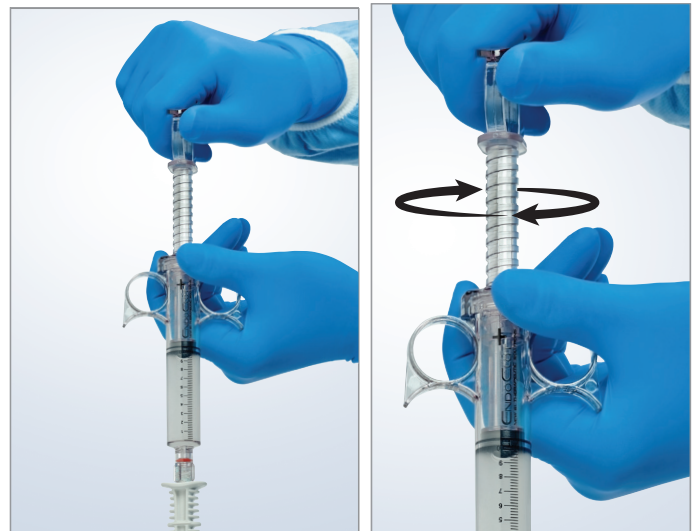

12. Wind plunger to inject agent as appropriate.

Olympus is a registered trademark of Olympus Corporation, Olympus America Inc., and/or their affiliates. | Medical devices listed may not be available for sale in all countries.

OLYMPUS

OLYMPUS AMERICA INC.
3500 Corporate Parkway, PO Box 610, Center Valley, PA 18034

For more information, contact your Olympus sales representative, or call 800-848-9024.

www.medical.olympusamerica.com
©2022 Olympus America Inc. All rights reserved.
Printed in USA OAIET0322ARG44644