


ESG-410

Sockets and screens

Quick Reference Guide

ESG-410


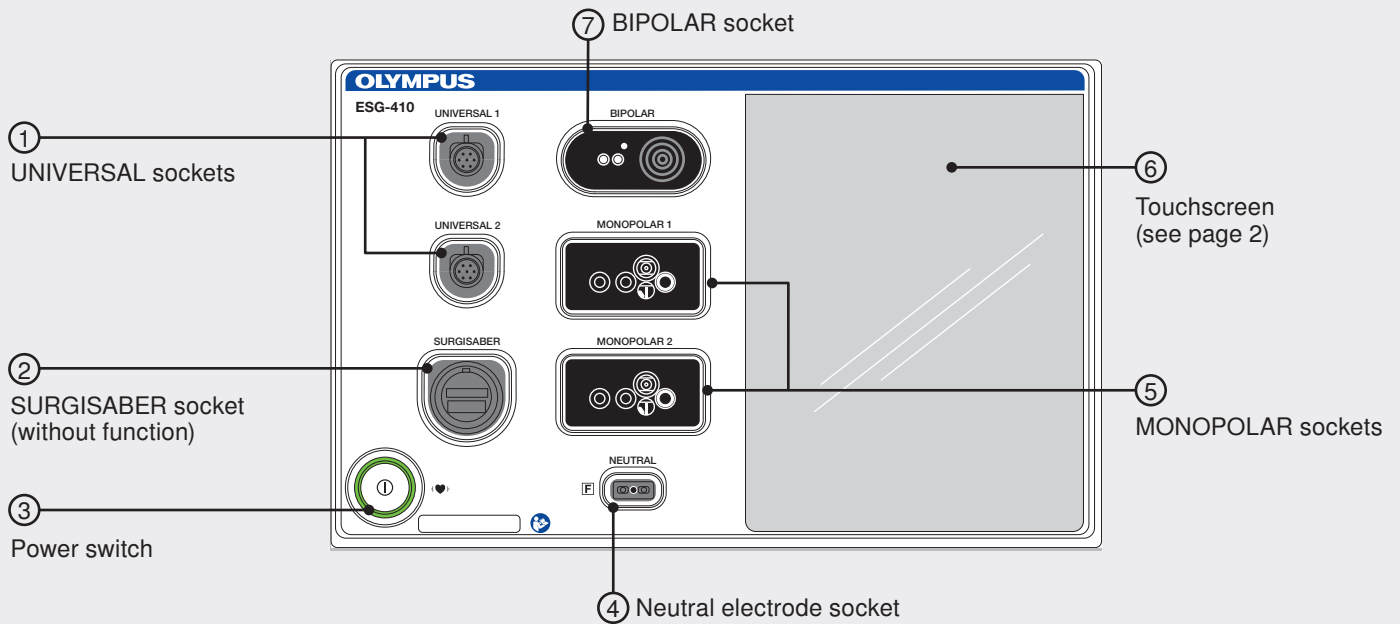


Instrument sockets and compatible plug types

Sockets

Instrument plugs


MONOPOLAR

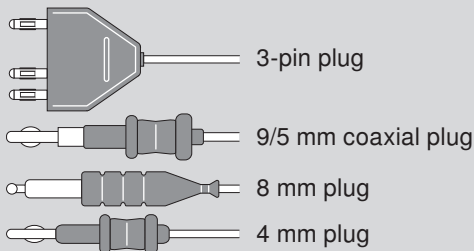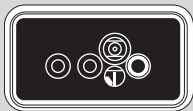

Monopolar instruments are not automatically recognized by the ESG-410.

⚠ WARNING
A 4 mm plug can only go into the white ring.

BIPOLAR

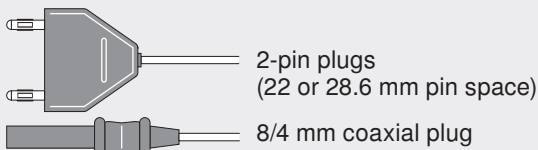

Bipolar instruments are not automatically recognized by the ESG-410.

UNIVERSAL



Only for Olympus instruments which are automatically recognized by the ESG-410.

Home screen

Procedure name

Autostart enabled

Socket or instrument name

Each of the 6 panes corresponds to 1 of the 6 sockets

CQM indicator
Shows the type of neutral electrode connected and the skin contact quality for split/dual neutral electrodes

Toggle switch
To switch the output socket on and off

Assigned foot switch
Shows the foot switch assigned to this socket (see page 3)

CQM info button
Tap this button to access baby mode (see page 4)

Is the patient a baby?

Turn on baby mode via CQM info

Set screen

Tap on a socket pane on the home screen (e.g. Bipolar) to adjust mode, power and effect for an instrument.

Autostart enabled

Mode

Power

Effect

Autostart On/Off button

Explanations of mode options

Autostart activation

HF output starts immediately on tissue contact when Autostart is enabled.

WARNING
Unintended contact can cause burns to the patient or OR staff.