

ESG-410

Foot switches, neutral electrodes and procedures Quick Reference Guide

Foot switches

1. Connect a wired foot switch or a receiver to the foot switch sockets on the rear panel of the ESG-410.

	Wireless	Wired
Rear panel of ESG-410		
Receiver		
Available foot switch types		
Foot switch indicators		

Does the receiver match the footswitch?

There is one receiver for every foot switch. The number stickers on both should be the same.

2. Tap the foot switch button on the home screen.
3. Tap the socket pane that requires a foot switch.
4. Once the selected socket is highlighted, tap the desired foot switch under [Select foot switch].

Foot switch

Select socket

- PK_CF_5mm
- Bipolar (Highlighted socket pane)
- Universal 2
- Monopolar 1 (Unassigned socket pane)
- Surgisaber
- Monopolar 2 (Socket with an assigned foot switch)


Select foot switch

- Indicators of connected foot witches
- Socket with an assigned foot switch
- No foot switch connected

Home

Neutral electrode setup screen

The [Neutral electrode setup] screen provides information on types and states of neutral electrodes.

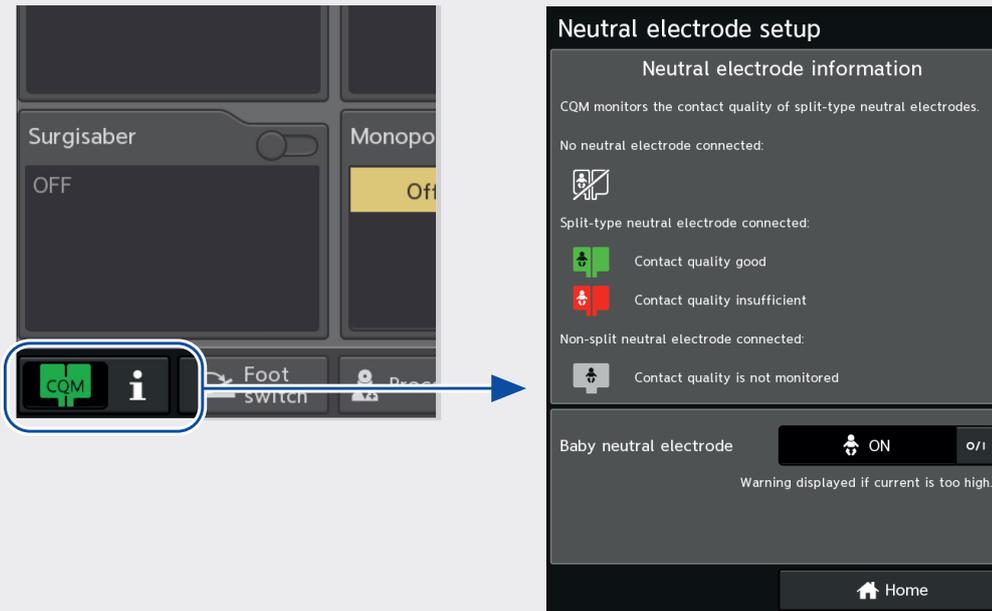


Contact quality will be shown for split type/dual pad neutral electrodes. Contact quality is not known for non-split type/single pad neutral electrodes and won't be indicated.

Is the patient a baby?

Turn on baby mode to receive a warning message when the current flow through the neutral electrode pad is too high.

Procedure screen

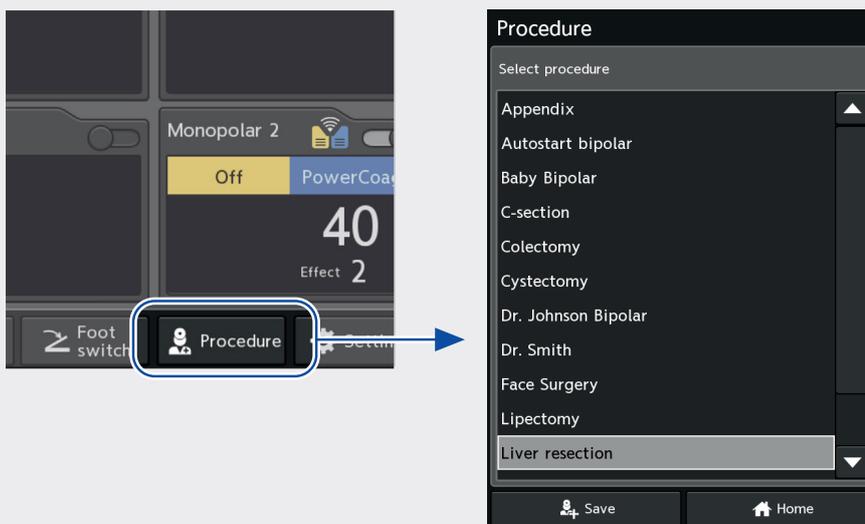

To load a saved procedure, select a procedure from the list.

To save the current settings, tap [Save]. This will save:

- Modes, power, effect
- Foot switch assignment
- Autostart state
- Baby neutral electrode state