

Reprocessing Flexible Endoscope Accessories

Manual Cleaning Prior to Sterilisation for the MAJ-2092

WARNING This workflow is a **SUMMARY** of the reprocessing steps necessary to properly reprocess your MAJ-2092. **Following disconnection and disassembly of the MAJ-2092 utilise the IFU for how to reprocess the MAJ-2092 and sealing accessories.**



MAJ-2092 (Luer-Split)

WARNING Be sure to remove the sealing accessory before reprocessing. Otherwise, it may not be reprocessed properly. (Follow the IFU for reprocessing instructions for the sealing accessory)



Precleaning

1 Disconnect from the endoscope




2 Completely immerse in detergent solution




Manual Cleaning


3 Completely immerse the Luer-Split (MAJ-2092) in detergent solution

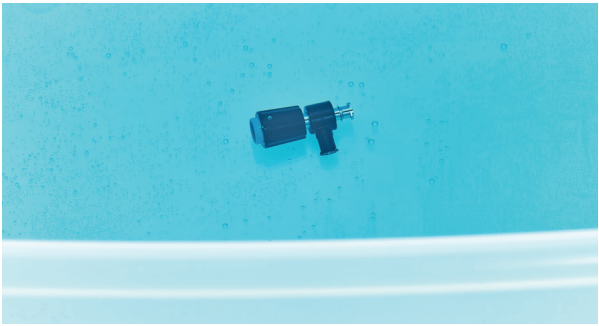

4 Wipe and clean the external surfaces. Brush the internal surface of EVERY part of the Luer-Split

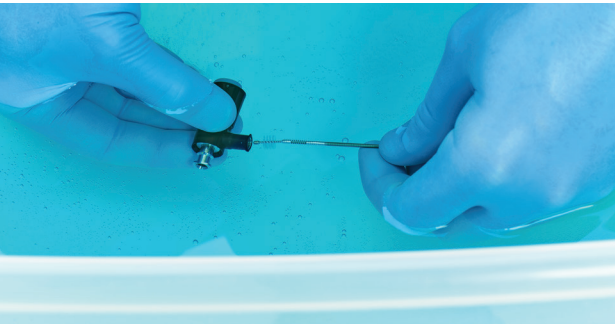


5 Flush the interiors, the openings, and the holes

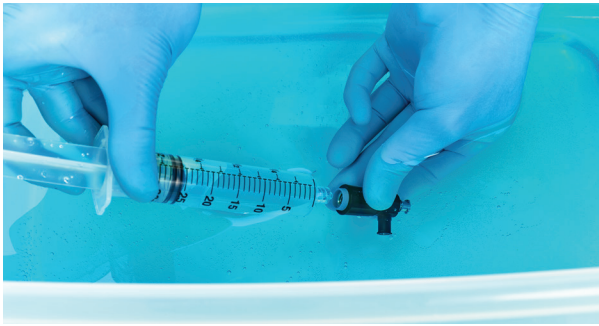

Rinsing

6 Thoroughly rinse

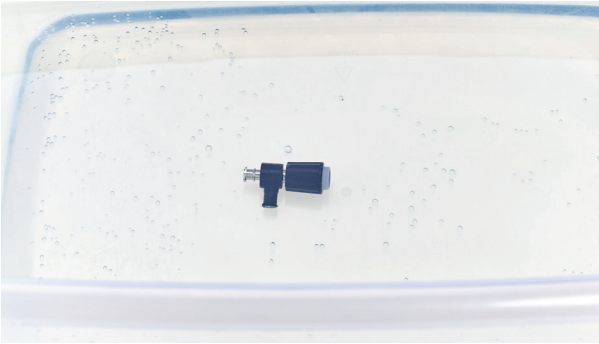


Drying

7 Immerse in Alcohol



8 Thoroughly dry

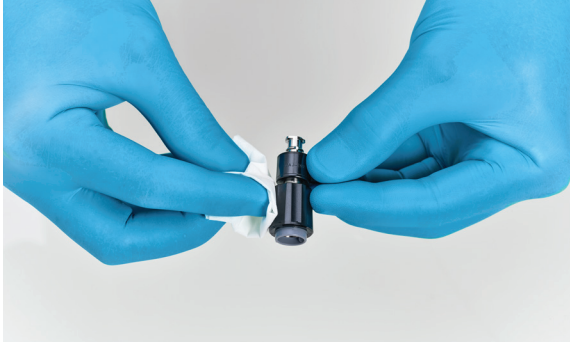

Note: Prior to sterilisation thoroughly dry the Luer-Split and seal in individual packaging appropriate for the sterilisation method chosen.