

EndoClot® Submucosal Injection Solution (SIS)

Quick Reference Guide

CAUTION: This guide is only a summary of the instructions for use for the EndoClot® Submucosal Injection Solution. Be sure to follow the instruction manual included in the device packaging.

Product inspection and components

- Inspect package for damage
- Peel off cover



Spiral dispensing syringe

Vial containing 0.5g (SIS-15)/1g (SIS-30)
AMP® powder

Plastic tray with holder for vial



Prepare the agent



1 Place vial in holder.



2 Inject 15ml (SIS-15) /30ml (SIS-30)
of sterile 0.9% saline into vial.



3 Mix by shaking vial.

EndoClot® Submucosal Injection Solution (SIS)

Quick Reference Guide

CAUTION: This guide is only a summary of the instructions for use for the EndoClot® Submucosal Injection Solution. Be sure to follow the instruction manual included in the device packaging.

Prepare the agent (continued)



4 Attach needle to SIS syringe.

Draw the agent into syringe



5 Unscrew locking cap to free syringe plunger.

Draw the agent into syringe (continued)



6 Pull the syringe plunger to draw agent.



7 Remove needle from syringe, then purge any air in the syringe.



8 Re-screw the locking cap.

EndoClot® Submucosal Injection Solution (SIS)

Quick Reference Guide

CAUTION: This guide is only a summary of the instructions for use for the EndoClot® Submucosal Injection Solution. Be sure to follow the instruction manual included in the device packaging.

Connect injection needle



9 Connect to needle and tighten the Luer-lock.



11 Pass the injection needle down the scope channel.



10 Purge air from needle by winding plunger.

Injection



12 Wind plunger to inject agent as appropriate.

As medical knowledge is constantly growing, technical modifications or changes of the product design, product specifications, accessories and service offerings may be required.

OLYMPUS

OLYMPUS EUROPA SE & CO. KG
Postbox 10 49 08, 20034 Hamburg, Germany
Wendenstrasse 20, 20097 Hamburg, Germany
Phone: +49 40 23773-0, Fax: +49 40 233765
www.olympus-europa.com