



MR Conditional

Retentia™ HemoClip

Quick Reference Guide


NOTE: This guide is only a summary of the instructions for use for the device. Be sure to follow the Instruction Manual included in the device packaging.

WARNING: Failure to follow the recommended MR Conditional labeling may result in a deployed GI hemoclip dislodging from tissue or heating of tissue at the GI hemoclip location. A GI hemoclip dislodgement may result in rebleeding – requiring additional intervention or surgery – serious injury, or death.

PRECAUTION: It is recommended that healthcare providers distribute patient implant cards with the name of the clip and date it was placed at the GI hemoclip location. A GI hemoclip dislodgement may result in rebleeding – requiring additional intervention or surgery – serious injury, or death.




1. Apply slight pulling pressure to the slider to keep the clip closed while it is being inserted into the scope.

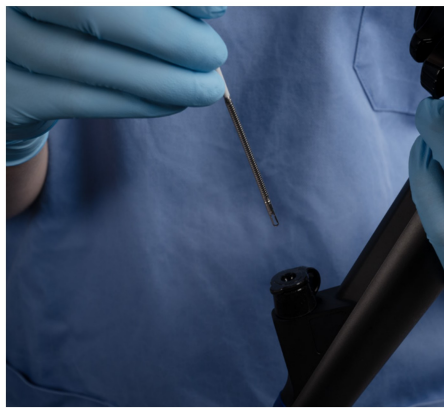

2. Once through the scope, push the slider to open the clip.



3. To rotate the clip, grip its base firmly. Rest the slider gently between your index and middle fingers, then rotate the base to adjust the clip orientation.

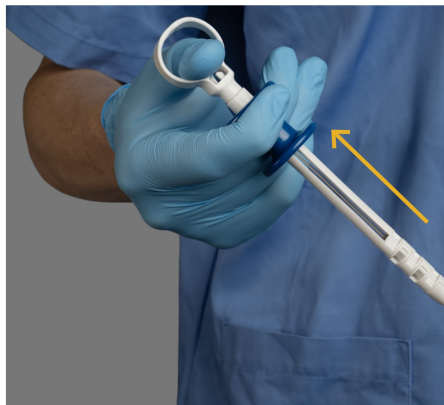

4. When the clip is in the desired position, pull the slider back to close and deploy the clip. You will hear an audible click once the clip is deployed.



5. Remove the sheath from the scope. As the sheath marking becomes visible, note that you are nearing the end of the device, and the distal part will withdraw.

Retentia Hemostasis Clip is manufactured by Yangzhou Fartley Medical Instrument Technology Co., Ltd. and distributed by Olympus.

Specifications, design and accessories are subject to change without any notice or obligation on the part of the manufacturer.

Olympus is a registered trademark of Olympus Corporation, Olympus America Inc., and/or their affiliates. | Medical devices listed may not be available for sale in all countries.