
Ultrasound Gastrovideoscope Cleaning Guide

GF-UCT140-AL5, GF-UCT180, GF-UE160-AL5

— Refer to companion *ENDOSCOPE DISINFECTION GUIDE* for disinfection information —

WARNING: This guide is only a summary of the steps necessary to properly clean your endoscope. Be sure to follow the detailed steps outlined in the *INSTRUCTION MANUAL* that was included with your endoscope purchase.

PRECLEANING — Immediately After Use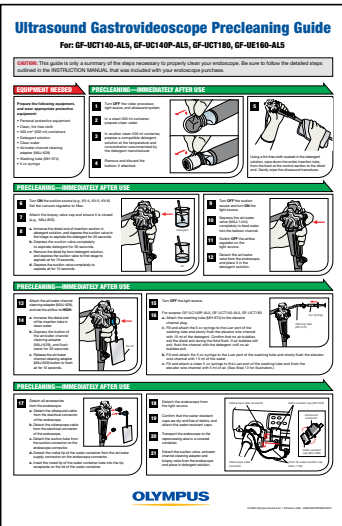


Important: Follow the steps outlined in the precleaning guide for this endoscope prior to manual cleaning.

LEAKAGE TESTING

CAUTION: If a leak is detected, the endoscope must be repaired. Follow the manufacturer's instructions.

Attach leakage tester. Turn on pump. Immerse endoscope in clean water. Perform leakage test (angulate tip). Remove from water. Detach tester from the air source. After the tip has deflated, detach the tester from the water-resistant cap.

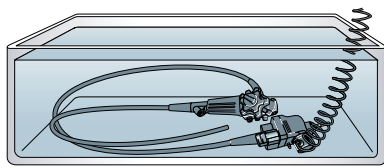MANUAL CLEANING

1

Immerse in freshly prepared detergent solution. Clean all external surfaces. Brush endoscope distal tip (and forceps-elevator, if applicable).

2 While immersed:

Lower forceps elevator and brush the interior groove of forceps elevator. Brush both sides of the instrument channel opening.

3 While immersed:

Raise the forceps elevator and brush both sides of the forceps elevator, including the opposite side of the groove.

MANUAL CLEANING

4 While immersed:

Move the elevator control lever in each direction 3 times.

5 Flush with detergent solution:

With the forceps elevator raised, flush the interior of the forceps elevator with 30 mL of detergent solution.

6 While immersed:

Using short strokes, feed the brush until it emerges from the distal end of the endoscope. Brush the insertion tube portion of the suction channel. Repeat until all debris is removed.

7 While immersed:

Using short strokes, feed the brush until it emerges from the suction connector on the endoscope connector. Brush the universal cord portion of the suction channel. Repeat until all debris is removed.

MANUAL CLEANING

8 While immersed:

Using short strokes, feed the brush until it reaches the distal end near the balloon attachment groove. Brush the balloon channel. Repeat until all debris is removed.

9 While immersed:

Brush the suction cylinder and instrument channel port. Repeat until all debris is removed.

10

Attach suction valve, suction adapter to the instrument channel port, and the suction tube from the suction pump to the suction connector on the endoscope. Turn the suction pump ON.

11

Depress and hold suction valve. Depress the suction valve to the first stage and aspirate detergent solution for 30 seconds.

MANUAL CLEANING

12

Immerse the endoscope's distal end in the detergent solution again and depress the suction valve completely and aspirate the detergent solution for 30 seconds.

13

Attach injection tube (MH-946) and channel plug (MAJ-621) and immerse suction port of the injection tube in the detergent solution. Inject 150 mL of detergent solution into the air/water channel and the balloon channel.

14 For endoscopes with elevators only:

Use the 5 mL syringe to inject 15 mL of detergent solution into the elevator wire channel via the washing tube. Flush detergent solution into elevator wire channel.

15

Wipe all debris from the endoscope's external surface. Soak the endoscope and all reprocessing equipment in the detergent solution for the time, temperature and concentration recommended by the detergent manufacturer.

RINSING

16

Rinse the entire instrument in clean water.

17

Flush 150 mL of clean water through the suction, balloon, and air/water channels. Flush 5 mL of clean water through the elevator wire channel (if applicable). Remove from water and repeat step by flushing all channels with air.

18

Use a clean, lint-free cloth to fully remove excess moisture from the endoscope and cleaning accessories in preparation for disinfection.

CAUTION

Do not hit the distal end of the endoscope against hard surfaces!

Do not forget to attach water resistant caps and plugs!

IMPORTANT

- Wear all appropriate personal protective equipment.
- Meticulous cleaning is essential for effective disinfection.
- Be sure to reprocess all removable parts (e.g., valves) according to the instruction manual.
- Visually inspect the equipment after cleaning. If debris remains, repeat the procedure.

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