

PolyLoop™ Ligating Device (HX-400U-30) Quick Reference Guide

The PolyLoop™ device delivers a nylon loop snare designed to prevent or control bleeding following polypectomy of pedunculated polyps.



CAUTION: This guide is only a summary of the instructions for use for the HX-400U-30 PolyLoop™ device. Be sure to follow the Olympus EndoTherapy PolyLoop™ HX-400U-30 instruction manual that was included with your HX-400U-30 purchase.

Preparation before using PolyLoop™ Ligating Device:



- 1 Remove the plastic retainer from the PolyLoop™ device and extend the plastic sheath until the PolyLoop is fully covered. Insert the PolyLoop through the working channel of the endoscope.

Instructions for using PolyLoop™ Ligating Device

- 2 Once in place, pull the yellow cylinder proximally until it stops to extend the PolyLoop™ device from the plastic sheath. Remove fingers from the yellow cylinder.

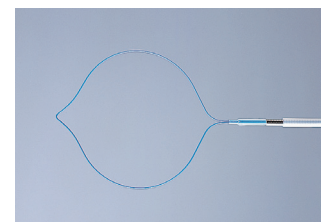
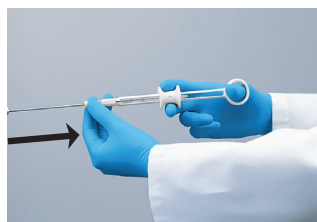


- 3 Position the PolyLoop™ device over the polyp at the base of the stalk, approximately 1-2 mm above the mucosa. Pull the slider towards the thumb ring to ligate the tissue. Look at the screen and feel the resistance in your hand to ensure you do not transect the polyp stalk.

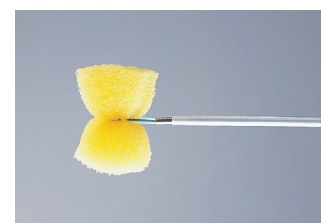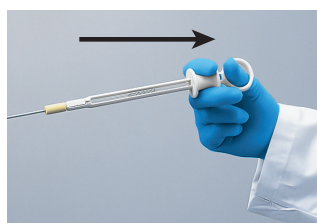

Caution: Do not apply unnecessary force to the loop when ligating tissue during the procedure.

- 4 With the yellow cylinder still touching the handle, push the slider to release the PolyLoop™ device.

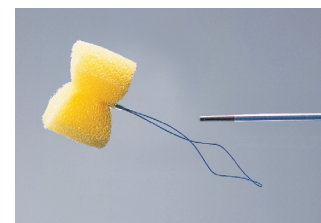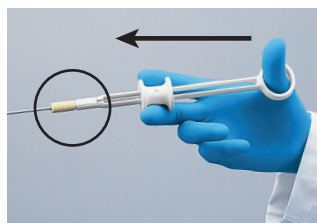

NOTE If the PolyLoop™ device gets stuck in the tube/coil sheath, use the Loop Cutter FS-410L/U to cut the PolyLoop™ device and to detach it from the device.

Refer to instructions on the reverse side for more details.



PolyLoop™ Ligating Device (HX-400U-30)


Quick Reference Guide

The PolyLoop™ device delivers a nylon loop snare designed to prevent or control bleeding following polypectomy of pedunculated polyps.

CAUTION: This guide is only a summary of the instructions for use for the HX-400U-30 PolyLoop™ device. Be sure to follow the Olympus EndoTherapy PolyLoop™ HX-400U-30 instruction manual that was included with your HX-400U-30 purchase.




Using an Electrosurgical Snare:

- 1 Look at the polyp for signs of ligation (blanching at the stalk, engorging in the head).
- 2 Close the snare at least 2 mm above the PolyLoop™ device on the stalk of the polyp. (See image at right.)

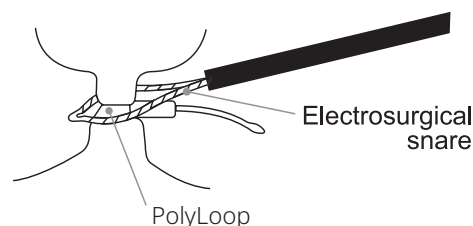


Releasing a Trapped PolyLoop Ligating Device:


- 1
 - Straighten the endoscope and PolyLoop™ device as much as possible. (See images A and B.)
 - Push the slider distally to extend the hook from the coil sheath. (See image C.)

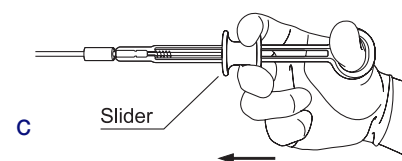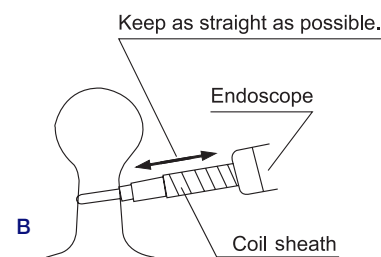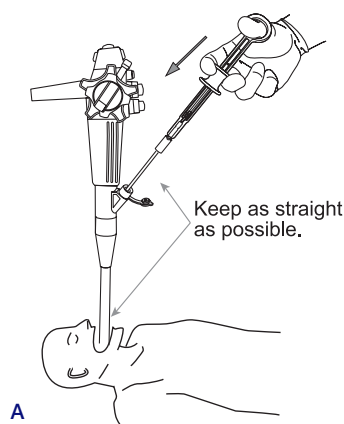

- 2
 - Cut the sheath, coil, and operating wire near the yellow cylinder using pliers or a wire cutter. (See image at right.)
 - Withdraw the endoscope.

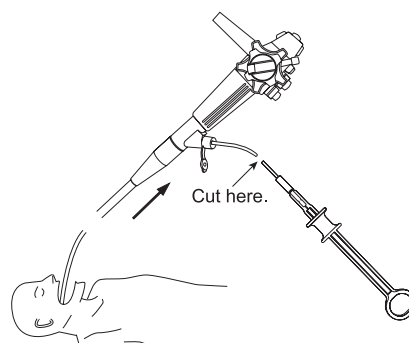

- 3
 - Insert the scope next to the sheath. Use the Loop Cutter to cut the PolyLoop™ device between the stopper and the coil sheath. (See image at right.)
 - Remove the sheath from the patient.

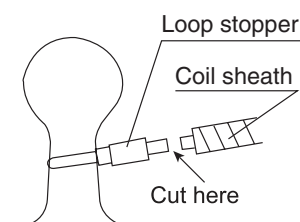

Olympus is a registered trademark of Olympus Corporation, Olympus America Inc., and/or their affiliates. I Medical devices listed may not be available for sale in all countries.

OLYMPUS

OLYMPUS AMERICA INC.

3500 Corporate Parkway, PO Box 610, Center Valley, PA 18034

For more information, contact your local Olympus sales representative, or call 800-848-9024.
www.medical.lympusamerica.com

©2021 Olympus America Inc. All rights reserved. Printed in the USA
OAIET0821QRG42269