ENDOSCOPE MANUAL CLEANING GUIDE

for EVIS X1™ (EZ1500/1100) & EVIS EXERA™ (190/180) Gastrovideoscopes & Colonoscopes

WARNING This workflow is a **summary** of the manual cleaning steps necessary when reprocessing EVIS X1 (EZ1500/1100) and EVIS EXERA (190/180) gastroscopes and colonoscopes. It does not include **precleaning/leakage testing/disinfecting/sterilizing** steps. Follow the detailed steps as instructed in the latest **ENDOSCOPE REPROCESSING MANUALS** for EVIS X1 (EZ1500/1100) and EVIS EXERA (190/180) gastroscopes and colonoscopes.

Before MANUAL CLEANING of the endoscope, **PRECLEANING & LEAKAGE TESTING** should be conducted as described in the reprocessing manual.

Steps 1 to 10 are performed while the endoscope is completely submerged in detergent or water.

1 Immerse in freshly prepared detergent solution. Clean all external surfaces


2 Brush the instrument and the suction channel*

3 Brush the suction cylinder & the instrument channel port**

4 Cover the suction cylinder to aspirate detergent solution through the instrument/suction channel

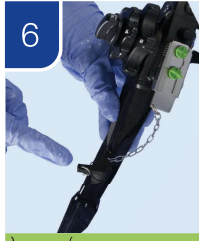6 Flush the air/water channel with detergent solution


7 Flush the auxiliary water channel with detergent solution (only applicable to endoscopes with auxiliary water feeding)

8 Wipe all the external surfaces again

9 Soak in detergent solution for the time recommended by the detergent manufacturer

10 While immersing in water, flush water through suction channel, air/water channel and auxiliary water channel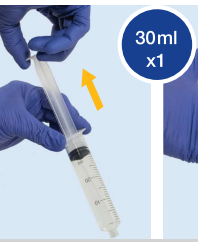

11 Flush air through suction channel and air/water channel three times and the auxiliary water channel once

12 Use clean lint-free cloths to dry the external surfaces of the endoscope

KEY

	Detergent Solution
	Water
	Air

* The channel cleaning brush part of the single use combination cleaning brush (BW-412T)

** The channel-opening cleaning brush part of the single use combination cleaning brush (BW-412T)

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